



SMART CAREGIVER CORPORATION



- 1 Year Warranty
  - Nurse Call Port
  - Locking Mounting Strap
  - Sensitive Equipment
  - Does Not Interfere with Magnet
- Non-Magnetic Pull-String Monitor**
- TL-01**

## Installation and Use Instructions

### TL-01 Non-Magnetic Pull-String Monitor

**CAUTION:** The manufacturer does not claim that this device will prevent falls and/or elopement. Pull-String mobility monitors are prone to difficulties. Be sure the cord cannot entangle or choke the user. Test this device before each use. This device is designed to augment caregivers' comprehensive resident mobility management program and is not a substitute for proper nursing care. The effectiveness of this monitoring device relies entirely on the direct supervision and immediate response by the caregiver to this monitor's alert. Failure to comply with this warning may result in injury or death. READ THE INSTRUCTIONS AND LEGAL DISCLAIMER.

**WARNING:** FAILURE TO COMPLY WITH WARNING MAY RESULT IN INJURY OR DEATH. This device is not suitable for all individuals. Other devices may be required. This device is not a substitute for visual monitoring by the caregiver. The manufacturer does not claim that this device will stop elopement and/or stop falls. This device is designed to augment the caregivers' comprehensive resident mobility management program.

**TEST THIS DEVICE BEFORE EACH USE. READ THE INSTRUCTIONS AND LEGAL DISCLAIMER BEFORE EACH USE.**


This Pull-String Monitor System is designed to be installed by the end-user. As such, it is the entire responsibility of the buyer to ensure that the system is properly installed and tested. Further, the system is not designed to replace good caregiving practices including, but not limited to:

- a.) Direct patient supervision
- b.) Adequate training for staff personnel for elopement and fall prevention
- c.) Testing of the system before each use


WHERE PERMISSIBLE, IT IS THE SELLER'S INTENT TO LIMIT ANY LIABILITY FOR SPECIAL, CONSEQUENTIAL OR INCIDENTAL ECONOMIC DAMAGES TO REFUND OF THE PURCHASE PRICE OR REPLACEMENT OF THE GOODS. SELLER DISCLAIMS ANY LIABILITY FOR COMPENSATORY OR OTHER DAMAGES ARISING OUT OF THE USE OF THE GOODS.

**LIMITED WARRANTY:** Seller warrants that the goods sold will be free from defects of workmanship and manufacturing, for a period of one year from the date of sale. Other than this limited warranty, the seller makes no express or implied warranties. Except for this limited warranty, NO EXPRESS OR IMPLIED WARRANTIES ARE GIVEN AND NO AFFIRMATION OF SELLER, BY WORDS OR ACTION, WILL CONSTITUTE A WARRANTY. THE GOODS BEING PURCHASED ARE BEING SOLD ON A "WITH ALL FAULTS" BASIS. THE ENTIRE RISK AS TO THE QUALITY AND PERFORMANCE OF THE GOODS IS WITH THE BUYER. SHOULD THE GOODS PROVE DEFECTIVE FOLLOWING THEIR PURCHASE, THE BUYER, AND NOT THE MANUFACTURER, DISTRIBUTOR, OR RETAILER, ASSUMES THE ENTIRE COST OF ALL NECESSARY INSTALLATION, SERVICING OR REPAIRS.

### TL-01 Non-Magnetic Pull-String Monitor

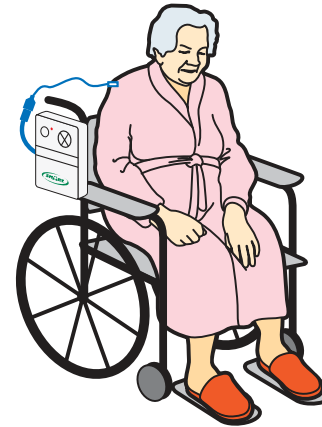

System in use, string attached to resident's clothing

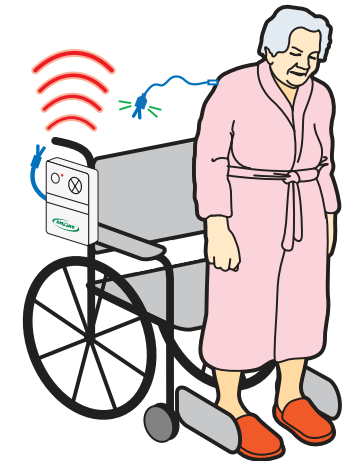

When the resident moves up and out of the chair, the separation switch will separate from the monitor, triggering the alert.

### About The TL-01 Non-Magnetic Pull-String Monitor:

Pull-string monitors reduce falls by alerting staff when residents attempt to depart from their chairs. The TL-01 does not interfere with magnet sensitive equipment.

#### CONTENTS:

- Pull-String Monitor
- Mounting Strap
- Adjustable Pull-String Cord



SMART CAREGIVER CORPORATION

1229 North McDowell Blvd., Petaluma, CA. 94954 | Tel: 800-650-3637


sales@smartcaregivercorp.com | www.smartcaregivercorp.com

2011-08

**Need Set-up Help? (800) 650-3637**


## 1 Install the Battery

Insert a new, 9 volt battery into the monitor as shown.

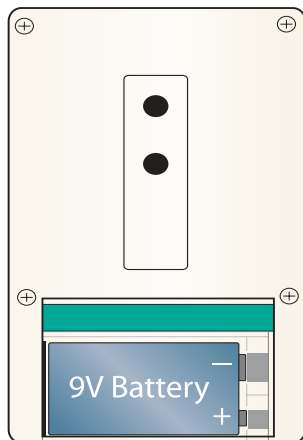


## 2 Mounting the Monitor

Attach the monitor to the handle or back of chair/wheelchair with the mounting clip and/or mounting strap provided. Make certain the monitor is securely attached and out of the resident's reach.

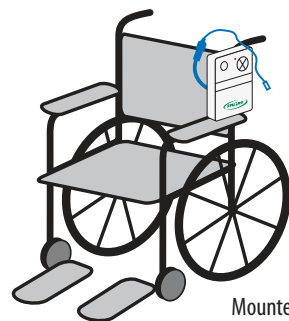


Mounted on Chair Handle

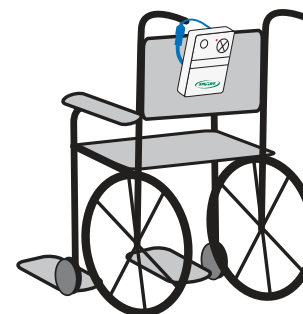


Mounted on Back of Chair

## 3 Test the System

With the monitor mounted on the chair, pull the string to separate the connection. The monitor will sound the alert tone.

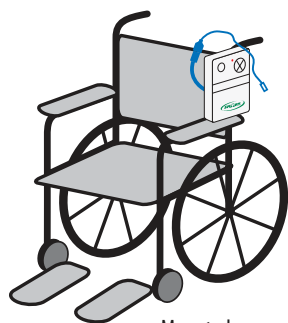

Mounted on Chair Handle

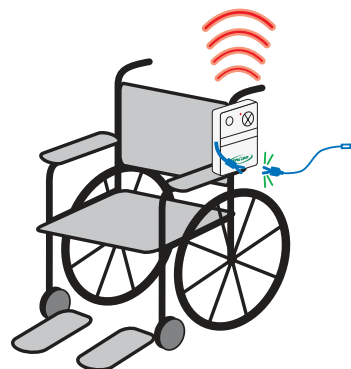


String Pulled and Monitor Alarming

## 4 Attach the Pull-String to the Resident's Clothing

The string should be routed to allow the resident freedom of movement. Securely attach the string to the resident's clothing. Make sure the string is not obstructed by any objects, such as cushions or clothing.

When the resident moves up and out of the chair, the connector will separate from the monitor triggering the alert.

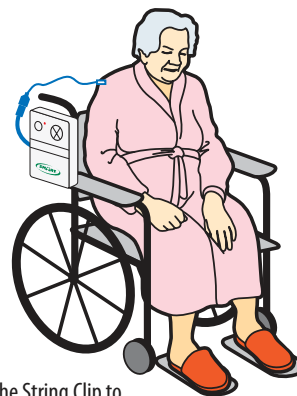

Attach the String Clip to Residents Clothing

